

# BodyGuard 323 Multi-Therapy Ambulatory Infusion Pump

## Multiply your Options for Care

**Deliver the widest range of care you'll find in one ambulatory infusion pump. Programmable for antibiotics, chemotherapy, TPN and PCA, the *BodyGuard 323* pump equips you with many therapeutic options.**

**The lightweight and compact, competitively-priced *BodyGuard 323* pump fits wherever you deliver care. You can use it pole-mounted at bedside. It's also easy for your patients to use at home.**



### Full Capabilities

#### Multiple Infusion Programs.

5 options for full treatment range:

- Continuous
- Intermittent
- Total parenteral nutrition (TPN)
- Patient-controlled analgesia (PCA)
- 25 pre-programmed steps

#### Integrated Pulse Oximetry.

A pulse oximeter plugs into the pump for continuous monitoring of SpO<sub>2</sub> and pulse rate.

#### Piston Mechanism.

Delivers ± 5 % accuracy and 1.8 µl linear infusion even at low flow rates.

#### 0.1 to 1200 ml/hr Flow Rates.

Extra-wide range to serve a variety of infusion regimens.

### Flexible Configuration

#### Inpatient/Outpatient Accessories.

A docking station and lockbox are available for pole-mounted use, and a carrying pouch for home care.

#### Rechargeable or Disposable Batteries.

Choose a rechargeable lithium-ion battery or standard 9V disposable batteries.

### Economy and Convenience

#### Money-Saving Rechargeable Battery.

A recent cost analysis estimates a savings of \$600.00 per pump per year using the rechargeable battery.\*



#### Uses Standard Medication Bags.

Reduce your cost of disposables.

### Safety and Security

#### BodyComm™ Downloadable

**History.** Review 24-hour patient history with charts and up to 1000 events log.

#### 3-Level Access Codes.

Prevent accidental or unauthorized pump program changes.

#### Ultrasonic Air Sensor.

Provides 2 levels of bubble detection.

# BodyGuard 323 Multi-Therapy Ambulatory Infusion Pump

<b>Pumping Mechanism</b>	Piston Pump
<b>Flow Rate</b>	0.1 to 1200 ml/hr <ul style="list-style-type: none"> <li>• in 0.1 ml increments up to 99.9 ml/hr</li> <li>• in 1 ml increments from 100 to 1200 ml/hr</li> </ul>
<b>Total Infused Volume</b>	0.1 to 9999 ml
<b>Flow Rate Accuracy</b>	± 5 %
<b>KVO Rate</b>	0.1 to 5 ml/hr (in 0.1 ml increments)
<b>Air Sensor</b>	Ultrasonic, adjustable 0.1 to 1 ml air bubble size (total air volume)
<b>Maximum Pressure</b>	Adjustable: <ul style="list-style-type: none"> <li>• 21 psi (high)</li> <li>• 14 psi (normal)</li> <li>• 7 psi (low)</li> </ul>
<b>Programmable Infusion Modes</b>	<ul style="list-style-type: none"> <li>• Continuous</li> <li>• Intermittent</li> <li>• Total parenteral nutrition (TPN)</li> <li>• Patient-controlled analgesia (PCA)</li> <li>• 25 pre-programmable steps</li> </ul>
<b>Power Supply</b>	110 – 240V AC, 50/60 Hz (20W / 0.12A)
<b>Battery</b>	3 Options: <ul style="list-style-type: none"> <li>• Lithium-ion 7.2V 1800 mAh (Rechargeable)</li> <li>• Extended-life lithium-ion 7.2V 3600 mAh (Rechargeable)</li> <li>• 2 alkaline 9 (Disposable)</li> </ul>

<b>Battery Life</b>	Rechargeable Battery: <ul style="list-style-type: none"> <li>• up to 15 hours at 125 ml/hr 1800 mAh</li> </ul> Disposable Batteries: <ul style="list-style-type: none"> <li>• 8 to 10 hours at 125 ml/hr</li> <li>• 12 to 14 hours at 90 ml/hr</li> </ul>
<b>Battery Charging</b>	<ul style="list-style-type: none"> <li>• Battery charges automatically when installed in charger connected to AC power source.</li> <li>• 6 hours required to charge fully-depleted battery.</li> </ul>
<b>Pump Alarms</b>	<ul style="list-style-type: none"> <li>• Air-in-Line</li> <li>• Occlusion (up/down)</li> <li>• End Program</li> <li>• Low Battery</li> <li>• End Battery</li> <li>• Pump Unattended</li> <li>• Door Open</li> <li>• System Malfunction</li> <li>• Lock Mode</li> <li>• Missing Key</li> </ul>
<b>Dimensions</b>	4.5 x 3.5 x 1.6 inches (H/W/D) (112 x 89 x 32 mm)
<b>Weight</b>	13.8 oz. (390 g) with standard battery
<b>Accessories (optional)</b>	<ul style="list-style-type: none"> <li>• Charger docking station</li> <li>• Lockbox</li> <li>• PCA bolus dose button</li> <li>• Wall charger</li> <li>• Carrying pouch</li> <li>• BodyComm™ history download system</li> </ul>



**Caesarea Medical Electronics GmbH.**  
 23 Staufenburg st. 72805  
 Lichtenstein, Germany  
 sales@cme-infusion.com  
 www.cme-infusion.com

International toll free  
 +800-323-57-500

